
SR-IOU-DISP


Instructions of Use

Disposable Sling Range

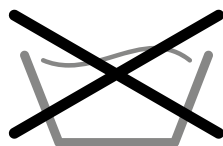

HIGH EASY LOOP FIXING SLING

DHEL




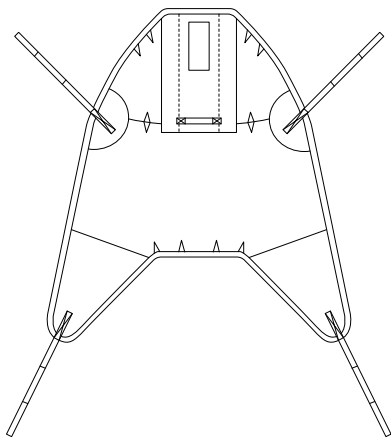


HIGH EASY CLIP FIXING SLING







DHEC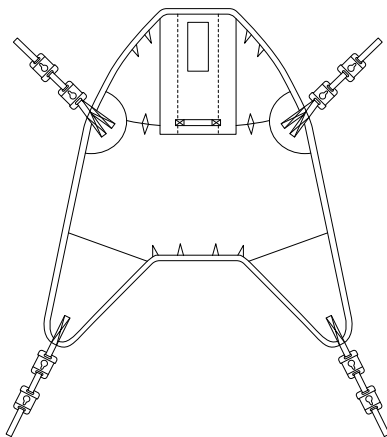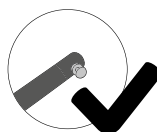

HIGH EASY LOOP FIXING SLING | DHEL

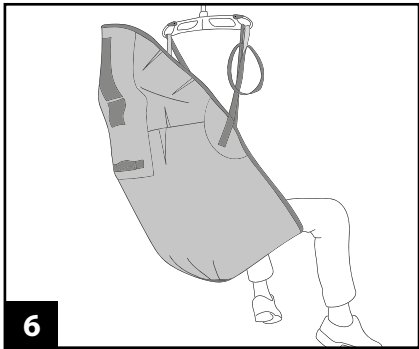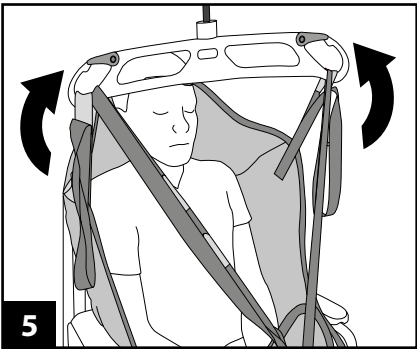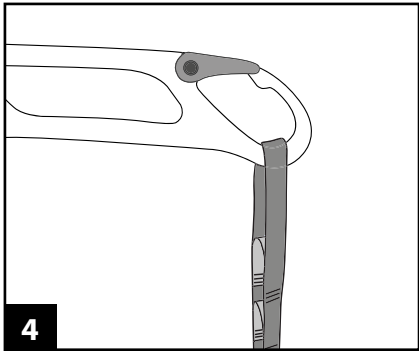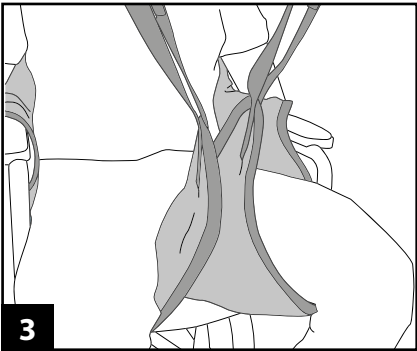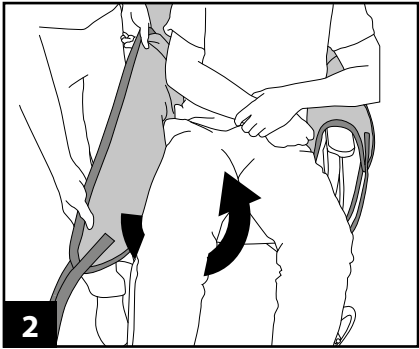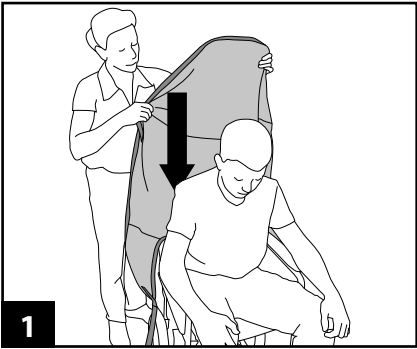High Easy Clip Sling | DHEC

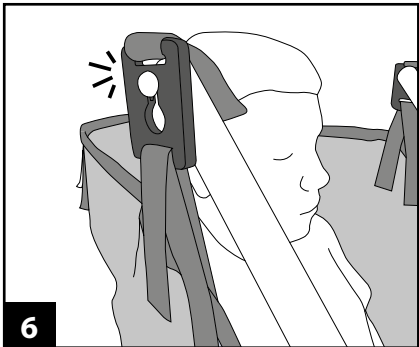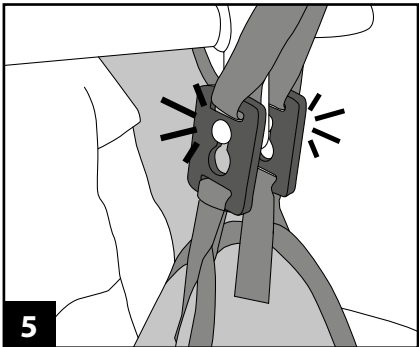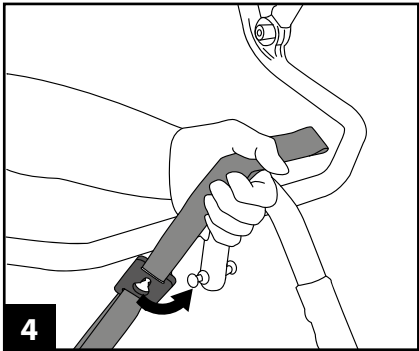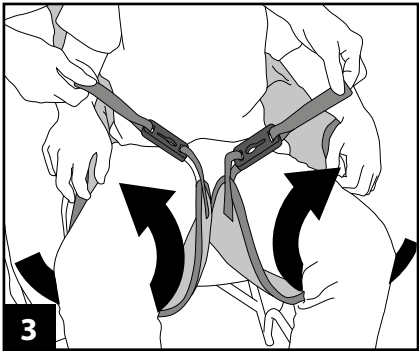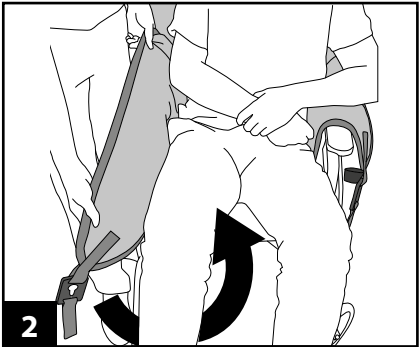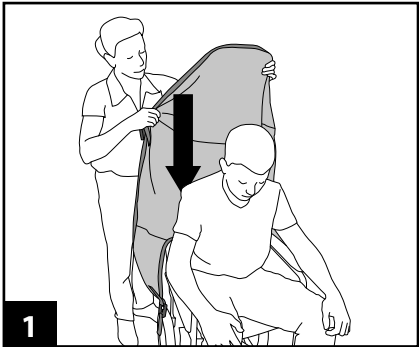For sizing guides and further information of this product please visit
www.silvalea.com

Or for further help and advice call
+44 (0)1626 331 655

Product Care Instructions

Important:

A risk assessment must be carried out to ensure that the correct size and style of sling has been met before use. While every attempt is made to instruct a safe transfer, you must conform to the current manual handling procedures/regulations. If you are in any doubt consult your Moving and Handling Advisor.

This is a short service sling and should be changed regularly as all non woven materials deteriorate quickly, therefore attention to the points below should be followed.

Service Label & Sling Checks:

All products should be checked each time they are used for any signs of deterioration.

Please Check:

- braiding/tapes for deterioration and fraying
- material for holes/snagging
- compatibility with the hoist
- safe working load of both the hoist and sling /product (if applicable)

Remove from use immediately if there are any signs of deterioration to the sling, tapes or label.

PLEASE NOTE: When the sling label below has come into contact with water and reveals a red warning sign, **REMOVE FROM USE IMMEDIATELY.**

The Silvalea disposable sling range is labelled with a Month & Year of manufacture.

If any serious accident has occurred in relation to any of the devices provided by Silvalea, please report immediately using the contact details provided and to the Competent Authority of the Member State in which the user/patient is established.
 (incidentteam@silvalea.com)

Washing Guidelines:

- Do Not Wash
- DO NOT TUMBLE DRY
- Do Not Use In A Damp Atmosphere
- DO NOT IRON
- DO NOT SMOKE
- SAFE WORKING LOAD 200kg (31st, 440lbs)

Product Care:

- Do not use the short service sling for bathing or showering a patient
- If the sling comes into contact with any liquid dispose of the sling immediately
- Do not use the sling if you can see deterioration of any kind
- Do not leave a patient in the sling unattended
- If in doubt, do not use the sling (Always ensure that the maximum load of the hoist is not exceeded).
- Silvalea recommends that all disposable slings are stored in a dry, controlled environment away from any moisture, damp and wetness and always within the original plastic outer bag.
- **It is strongly recommended that the Silvalea disposable slings are not stored for longer than a 5 year period from date of purchase – after which time they should be destroyed and not used.**

Disposal of the Sling:

- Dispose of the sling if it has become soiled, damaged or contaminated.
- This sling has been designed specifically for individual patient use – destroy the sling once the patient has been discharged in accordance with the local requirements and regulations.
- If sling has been used on a patient with an Infectious condition please refer to your infection control procedure

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
Revision: 04/01/2022

Silverhills Buildings, Decoy Ind. Estate,
Newton Abbot, Devon, TQ12 5LZ, UK

ADVENA Ltd. Tower Business Centre, 2nd
Floor, Tower Street, Swatar, BKR 4013. Malta

Silvalea Ltd a division of **savaria**. Copyright 2021 © All rights reserved

Copyright & Design Rights: Silvalea. Silva-Quick-2-Apply® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Superfine® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Safety Slot® is registered trademark of D-ansermed Ltd. All rights reserved. COOLMAX® is a registered trademark of the LYCRA Company.

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Silvalea Ltd silvalea.com