
Instructions of Use

Rehabilitation Sling Range

 Mesh


 Polyester

**Not all fabrics available with all styles*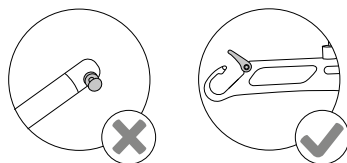

RACHEL HARNESS


 M921535 (040)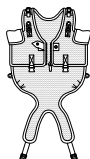

WALKING HARNESS


 WLKH MAL

 WLKH FEM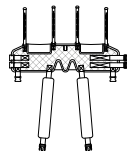


SILVA JOLLY JUMPER

 SILJJ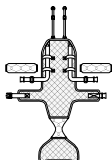


SUPPORTIVE LIFT PANTS

 SIL1129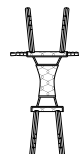

SIT TO STAND SLING

 405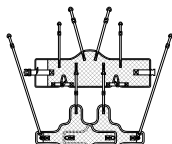

STAND AIDS

 D202

 202

 AS116


 AS112

 SIL300

 SIL114


 SILSB360

 BLISS
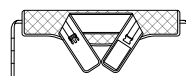


TRANSPORTERS






 204

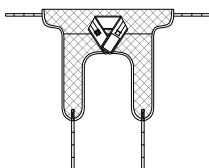




Rachel Harness Application






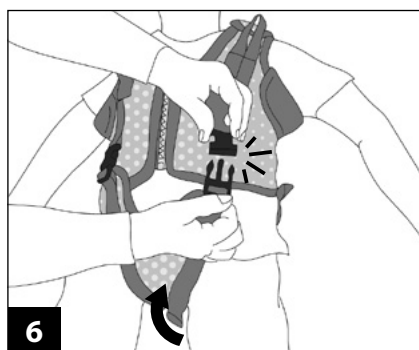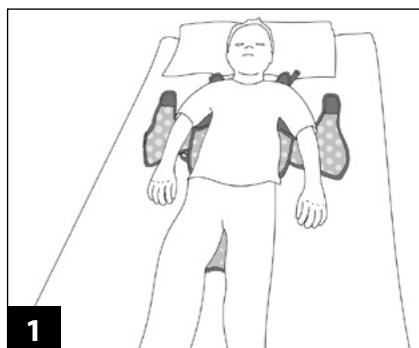
Rachel Harness Application






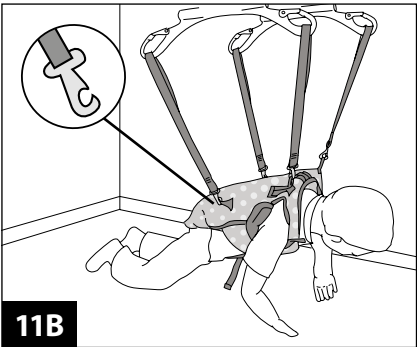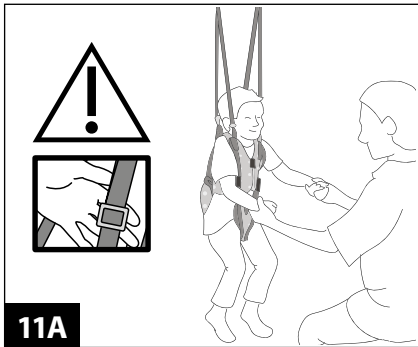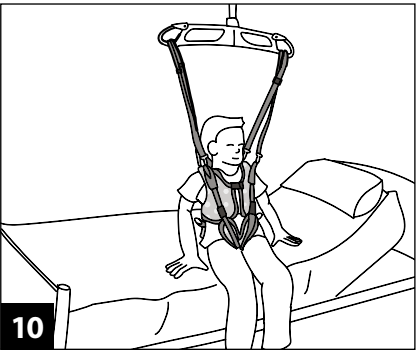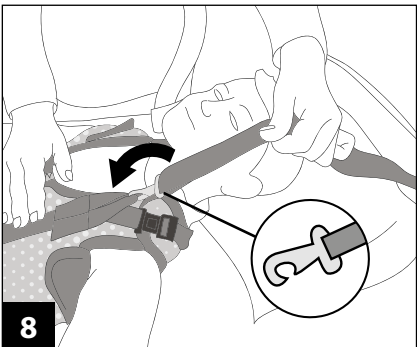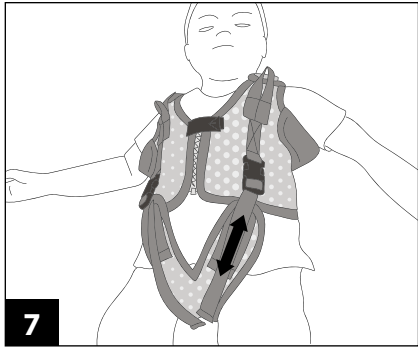
Silva Jolly Jumper Application






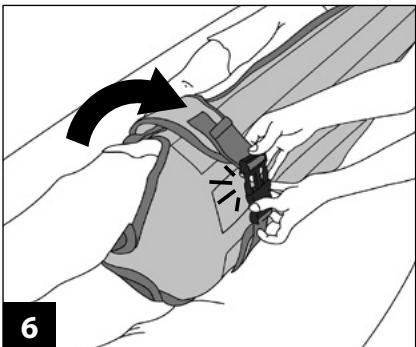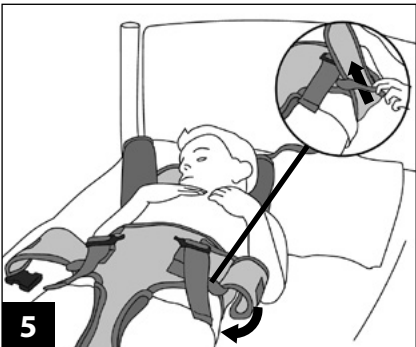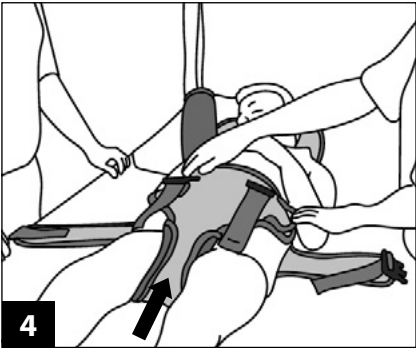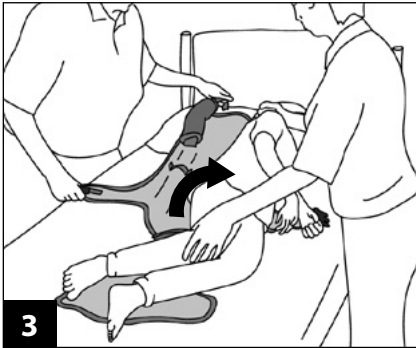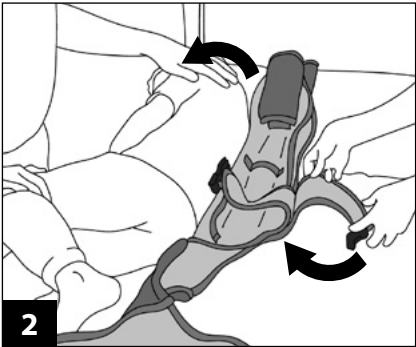
Silva Jolly Jumper Application






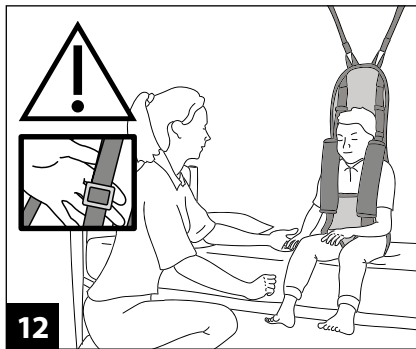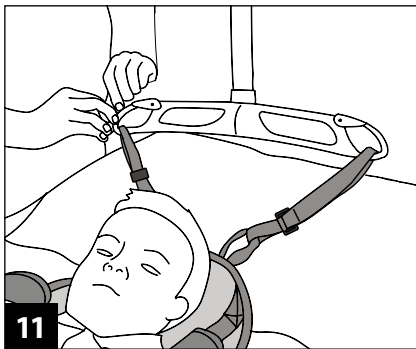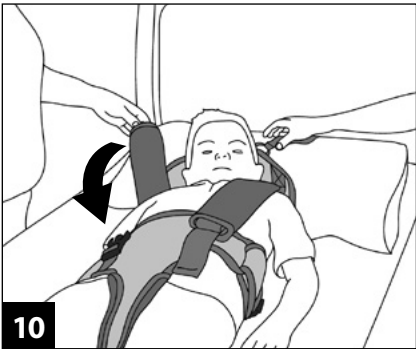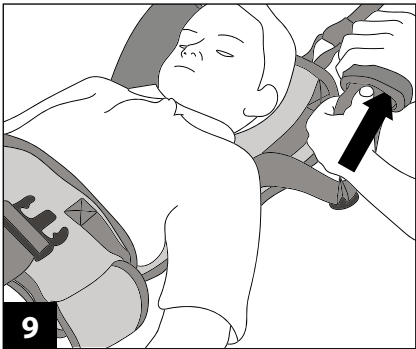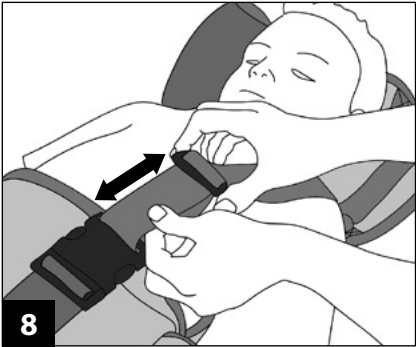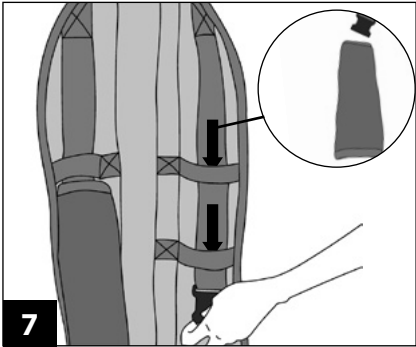
Male Walking Harness Application






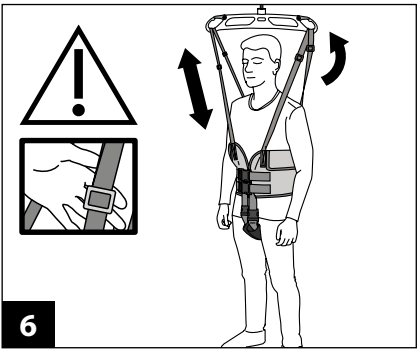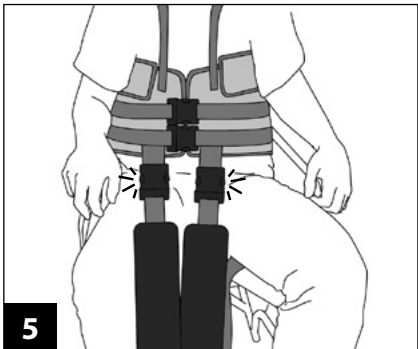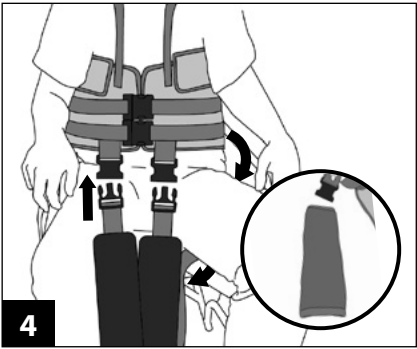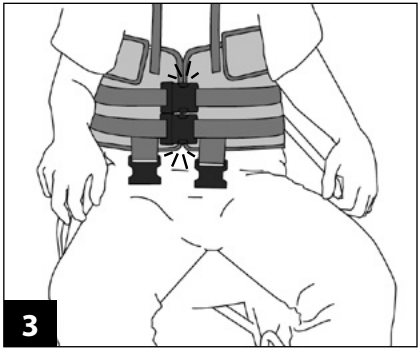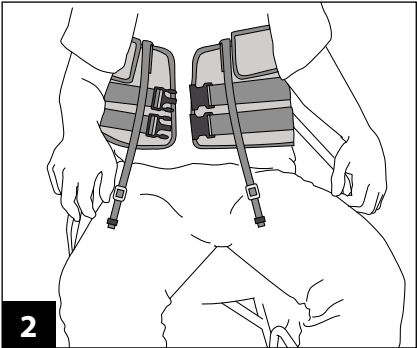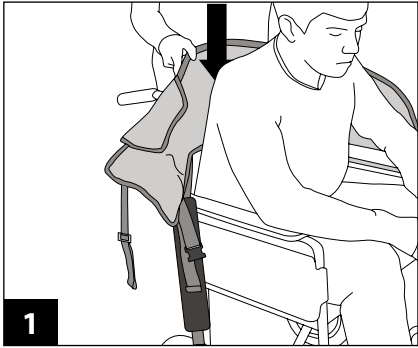
Female Walking Harness Application





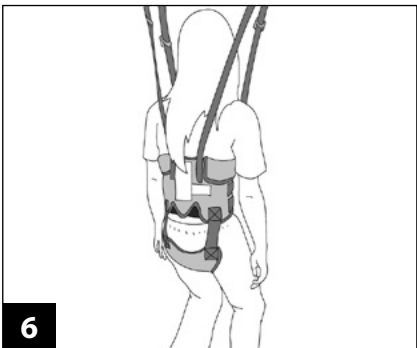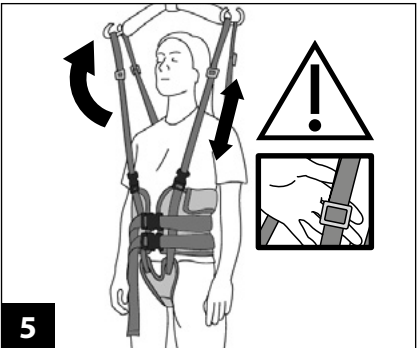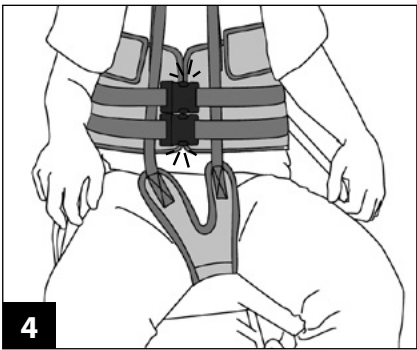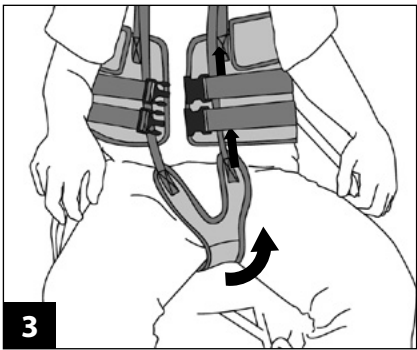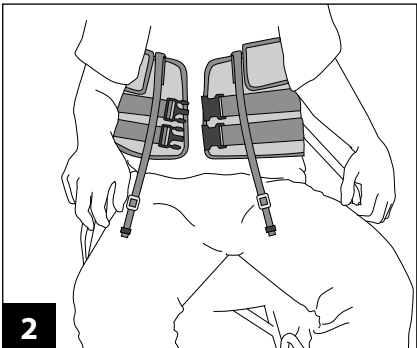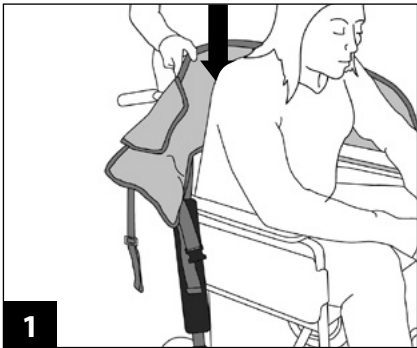
Supportive Lift Pants Application






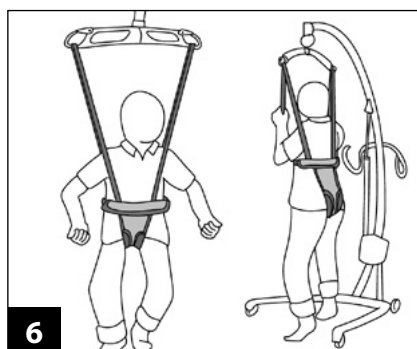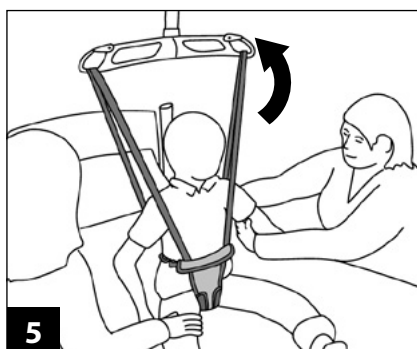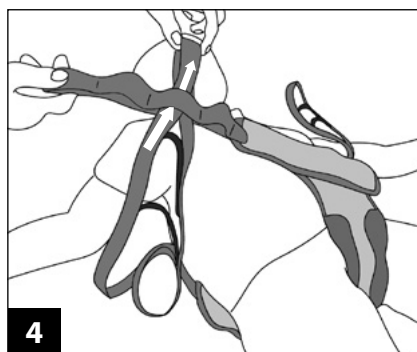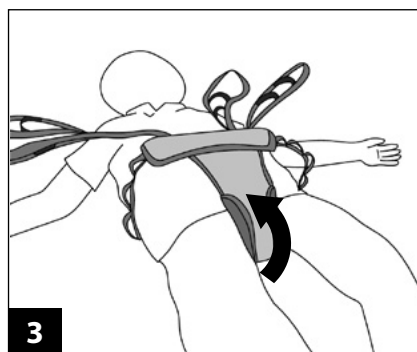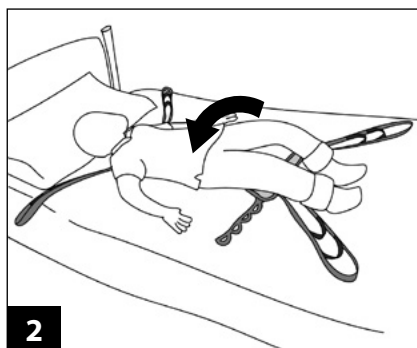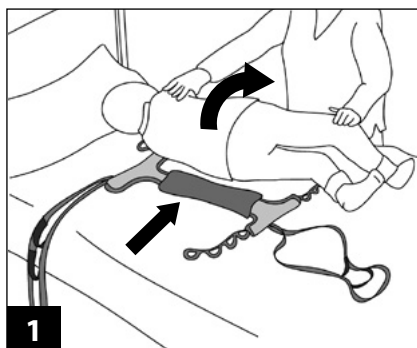
Sit to Stand Sling Application

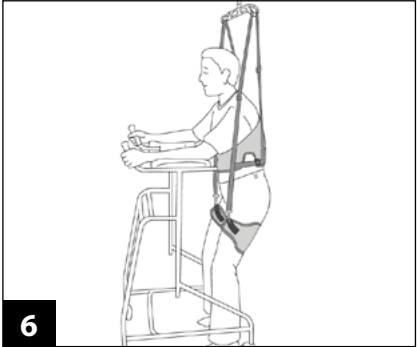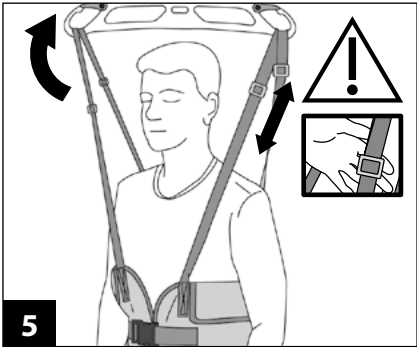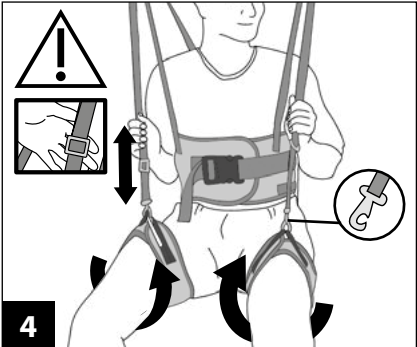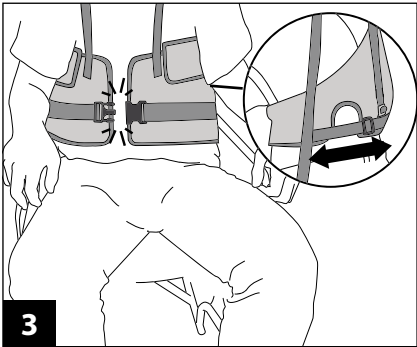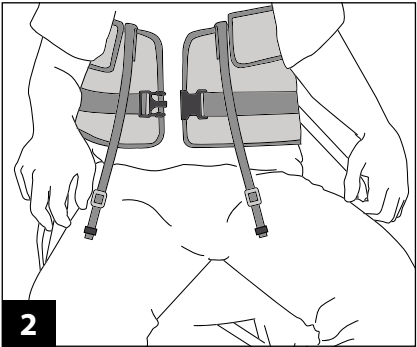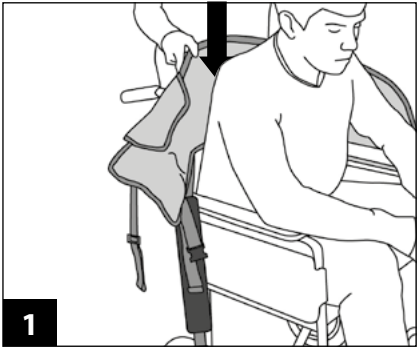Transporters Application

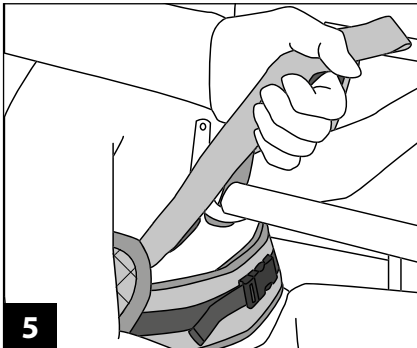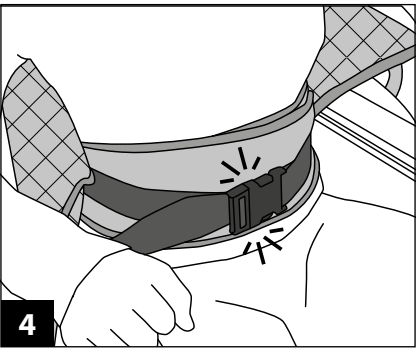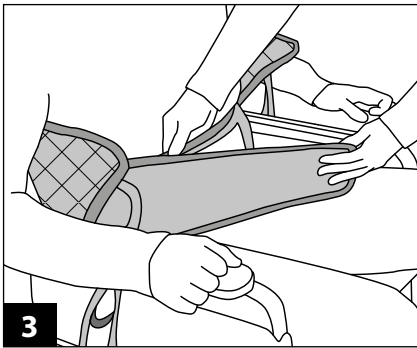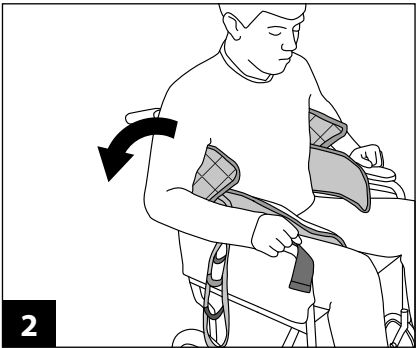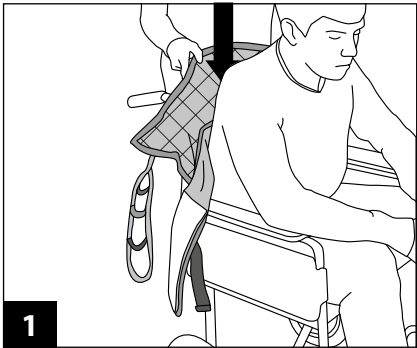Stand Aids Application

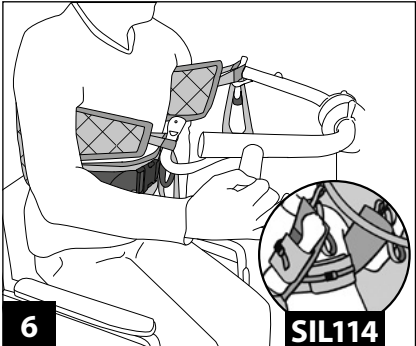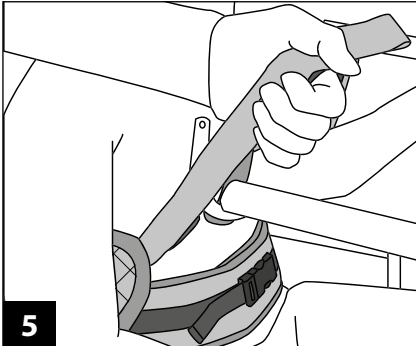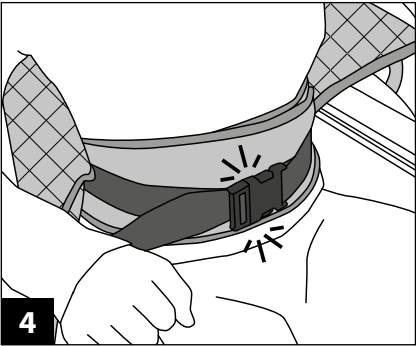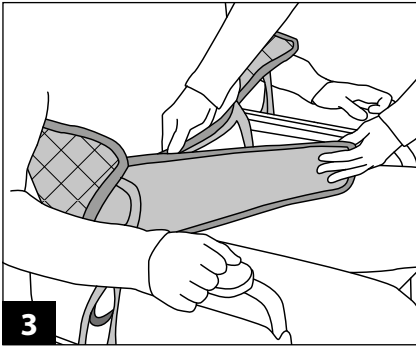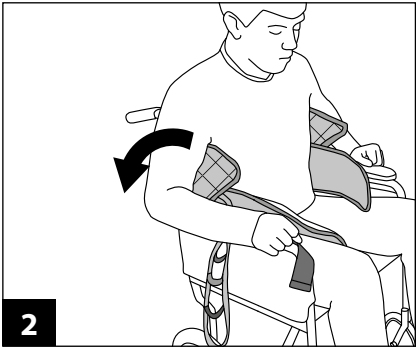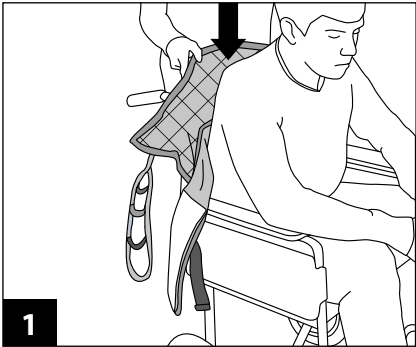Insurance Statement

Savaria Group products are always manufactured under strict safety protocols and confirm that products manufactured are interchangeable. The interchangeability of slings across different lift manufacturers should **NOT BE ASSUMED**. Please see our full statement on our website at: silvalea.com/downloads

Important

READ BEFORE USE.

This sling is designed for a general purpose transfer from a recumbent or seated position. If it is an In-Situ sling, the sling can remain in position under the patient subject to a risk assessment. Transfer accessories are intended to assist patient transfer.

The patient must be assessed by a competent person who is fully trained in the suitability, application and fitting of the sling. Carers should always be trained in the use of hoists and slings.

Requirements of LOLER state that a thorough examination should be carried out before first use, unless there is a valid Certificate of Conformity (C of C) made less than 12 months earlier. The C of C can be found on the despatch note and a copy can be requested.

Correct sizing of a sling is important, if in any doubt consult your Moving and Handling Advisor. Incorrect product or use could result in serious injury.

Do not attempt to lift the patient until all appropriate fixings are secured to the hoist spreader bar and you are satisfied that the sling is the correct style, size and fit. Correct sizing of sling is important as a sling that is too large could cause a patient to slip through the aperture. Always follow any care plan that might be in place for your patient.

It is advisable to use a sling with a head support when lifting from a supine position and if fitting from a bed – always raise the bed to waist height to minimise stress on the carers back. Mis-use of the product can result in injury.

Silvalea products are LATEX FREE.

Note: Clients with neurological deficits, such as dementia and Traumatic Brain Injury (TBI) can significantly impact a person's ability to follow directions, and motor and sensory functions.

Whilst every attempt is made to instruct a safe move you must conform to the current moving and handling procedures/regulations. If in any doubt, consult your moving and handling advisor.

Service Label and Sling Checks

All products should be checked each time they are used for any signs of deterioration.

Please Check:

- braiding / tapes for deterioration and fraying.
- material for holes / snagging.
- compatibility with the hoist.
- safe working load of the hoist and sling / product (if applicable).
- label is legible.

Remove from use immediately if there are any signs of deterioration to the sling or tapes or the label is illegible.

The Service Label is a visual indicator of when the 6-monthly or annual hoist inspection has been completed. This is in line with LOLER (Lifting Operations and Lifting Equipment Regulations).

Safe Working Load (SWL):

- is clearly stated on the sling label, this can vary depending on sling type always check label before use.
- SWL shall be clearly stated on the sling and hoist labels – never exceed the SWL on either the sling or the hoisting equipment.

Expected Life Span

- Before each use always refer to **Sling Checks**.
- Withdraw from use immediately if there are any signs of deterioration.
- Overtime excess laundering may affect the integrity of the product.
- Incorrect use of detergents (see washing guidelines) will accelerate the deterioration of the product.
- Mis-use of product will affect the expected service life.
- In-Situ (leave in products) - expected life span 18 months from the first use (this is only a guideline).
- Standard products - expected life span 24 months from the first use (this is only a guideline).

Disposal of Product

- Product should be disposed of within the organisation's waste management guidelines.
- Please recycle where possible.

Warranty

Silvalea offers a 12 month manufacturers warranty on all products unless otherwise stated. For more information, visit our website at: silvalea.com/downloads

If any serious accident has occurred in relation to any of the devices provided by Silvalea, please report immediately using the contact details provided and to the Competent Authority of the Member State in which the user/patient is established.

Product Care Instructions

ENG

Product Care

- Ensure that any 'hook and loop' fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the 'hook and loop'.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions / adaptations made to our products that have **NOT** been carried out by ourselves. Doing so may cause serious injury or death to the user.

Washing Guidelines

- Follow laundry temperatures for washing and drying detailed on the product label. (please note temperatures advised are a maximum).
- Launder separately to prevent cross contamination.
- Always ensure that all 'hook and loop', 'clunk-click' and 'buckle' fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- Use of bleach (or whitening agents), stain removers or fabric conditioners may harm the structure of the sling. The sling should only be cleaned in the recommended way.
- Discolouration may indicate fabric degradation.
- Metal fixings are sometimes used in our products. Care should be taken to ensure that all products containing metal attachments are washed in a pillowcase or wash bag to prevent damage to the product or to washing equipment. Silvalea accept no liability for damage to laundry equipment.
- Always remove the head support bones prior to washing to prevent damage to the sling and bones.
- Silva-Superfine with COOLMAX® mesh slings are provided with a wash bag. The wash bag **MUST** be used during laundering.
- **DO NOT IRON**

Drying Guidelines

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.

Testing

Sling Fabric Bench Testing

In-Situ Fabric Bench Testing report now available to download from our website at: silvalea.com/downloads

Instruction Symbol Key

Bed Application


Loop Hoists Only

Chair Application

Manual Handling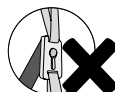

Upside Down Clip!

Minimum Four Person Lift



Clip Hoists Only

Wipe Clean

D'Anser Clip

Prior to use, check all clips are attached correctly.

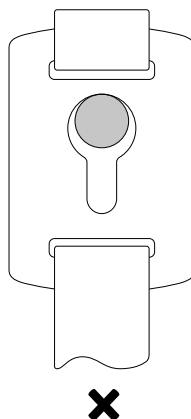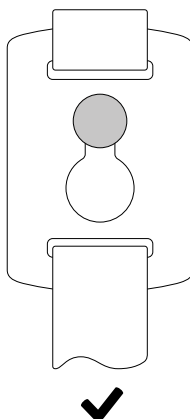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
For sizing guides, images and further assistance on these products, please visit: **www.silvalea.com**

For help and advice, contact: **customerservices@silvalea.com** or **+44 (0)1626 331 655**

Silvalea Ltd, Silverhills Buildings,
Decoy Industrial Estate, Newton Abbot,
Devon, TQ12 5LZ, United Kingdom

Registration Number: 04350939 | VAT Number: GB797412786

Silvalea Ltd, a division of  **savaria**.

ADVENA Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Copyright and Design Rights:

COOLMAX® Ecomade is a registered trademark of The Lycra Company.

COOLMAX FreshFX® is a registered trademark of The Lycra Company.

Silva-Superfine® is a registered trademark. Silvalea is the exclusive licensee of the trademark.

Silva-Safety Slots® is a registered trademark. Silvalea is the exclusive licensee of the trademark.

Silva® is a registered trademark. Silvalea is the exclusive licensee of the trademark.

Silvalea® is a registered trademark. Silvalea is the exclusive licensee of the trademark.